

SPECTRE

Wireless Hand Switch Foot Switch System
For GE-OEC™ Series 9800 & Series 9900 Mobile C-Arms

No more
broken cables!



- Compatible with GE-OEC Series 9800 & 9900 C-Arms
- Plug and play connectivity
- Encrypted transmission
- Factory mated units – Invisible to other devices
- One year full replacement warranty
- FDA 510k & ETL approved



SPECTRE

Operation:

The SPECTRE wireless foot and hand switch system is designed for use with GE-OEC Series 9800 and Series 9900 Mobile C-Arms. Both the hand and foot switch function like the original wired foot switch and hand switch, but wirelessly. Unlike the OEM wired foot switch, the SPECTRE foot switch includes a "save" button.

Installation:

The SPECTRE is plug and play. Simply connect the SPECTRE receiver where your existing hand switch is connected and it's ready to operate. (See photo below)

Size:

Foot switch: 2.8 lbs. Dimensions: 12.5" x 8" x 2.25"
Hand switch: 4 oz. Dimensions: 5.8" x 2.3" x .85"
Receiver unit: 4.5" x 4.5" x 2"

Battery:

The UL approved lithium ion batteries included with the foot switch will operate continuously up to 48 hours on a charge. The foot switch recharges in four hours. The hand switch will operate continuously up to 48 hours and recharges in four hours.

Encrypted and paired for security, safety:

Upon power up the system scans seventy communications channels for current use and selects only those that are not in use. The transmissions are also fully encrypted and factory paired, The SPECTRE hand and foot switch will connect only with a paired receiver and are invisible to all other un-paired devices, insuring total data integrity and interference-free use.

The SPECTRE system uses the 2.4 GHz bandwidth which is outside of all the assigned hospital equipment frequency spectrums. The Spectre uses spread spectrum frequency hopping which selects 1 of 70 sub-channels in which to operate.



Spectre wireless receiver

Range:

Up to 50 feet

Hand Switch/Foot Switch:

The hand switch and foot switch perform the same functions: Fluoro on, Save, One shot, and Mode Select. A hand switch storage/charger is included.

Warranty:

- One year full replacement warranty.



Certifications: ETL
Made in USA. Technical specifications subject to change.
OEC is a registered trademark of the General Electric Company.

TMI-2013-004D

Manufactured by:
TransAmerican
Medical Imaging

TransAmerican
Medical Imaging
400 N. Geneva Rd.
Suite E
Lindon, UT USA
84042
Tel: 800.865.8195
transamericanmedical.com



Watch the Spectre video

TRANSAMERICAN
Medical Imaging